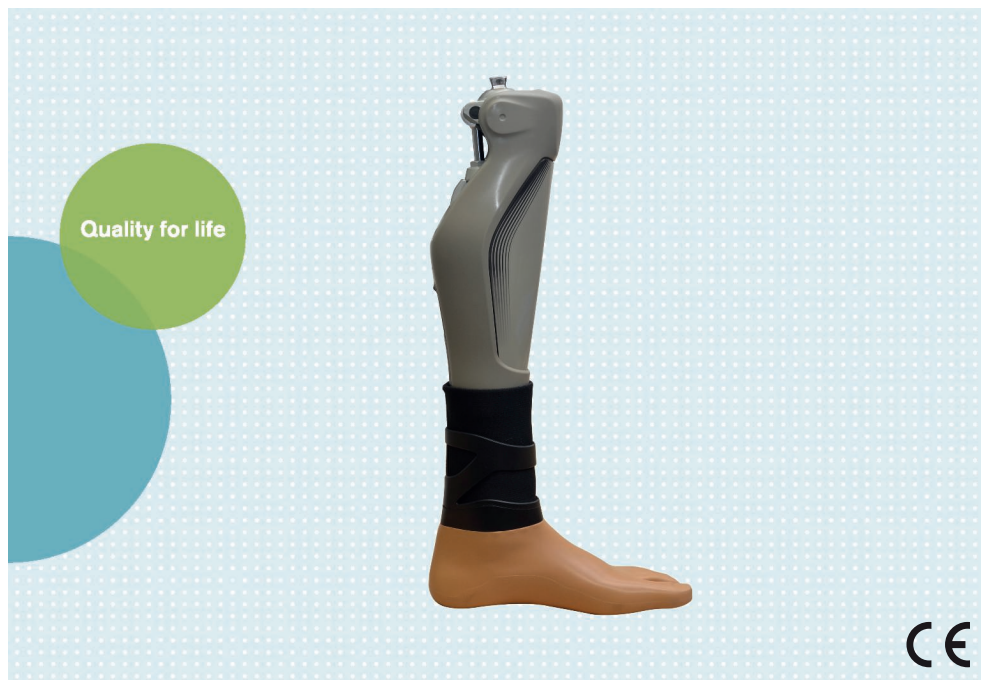
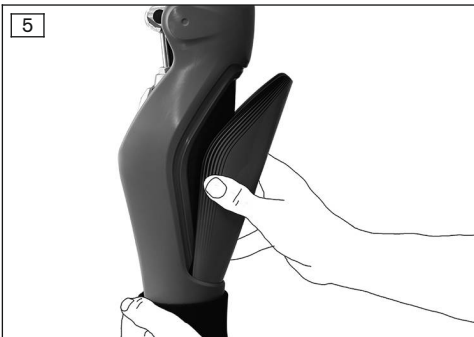
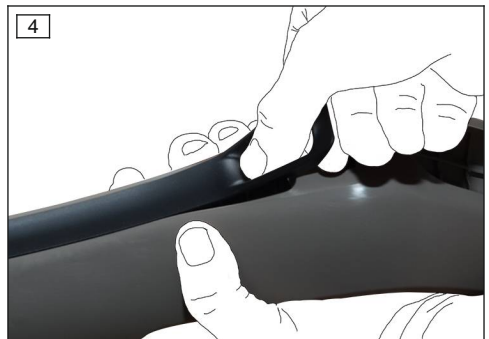
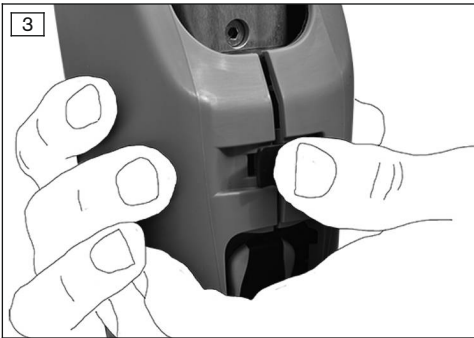
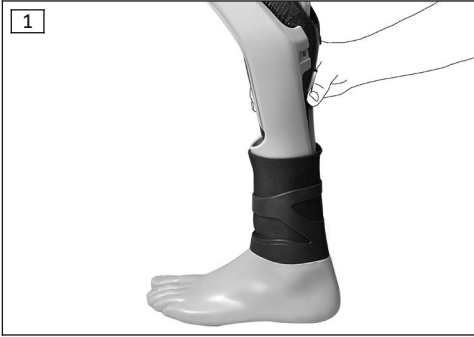
C-Leg Protector 4X860

DE Gebrauchsanweisung (Benutzer)	3
EN Instructions for use (user)	8
FR Instructions d'utilisation (Utilisateur)	13
IT Istruzioni per l'uso (Utilizzatore)	18
ES Instrucciones de uso (Usuario)	23
PT Manual de utilização (Usuário)	28
NL Gebruiksaanwijzing (Gebruiker)	34
SV Bruksanvisning (Användare)	39
DA Brugsanvisning (Bruger)	44
NO Bruksanvisning (Bruker)	49
FI Käyttöohje (Käyttäjä)	53
PL Instrukcja użytkowania (Użytkownik)	58
CS Návod k použití (Uživatel)	64
HR Upute za uporabu (Korisnik)	69
TR Kullanma talimatı (Kullanıcı)	73
EL Οδηγίες χρήσης (Χρήστης)	78
RU Руководство по применению (Пользователь)	84
ZH 使用说明书 (用户)	90
KO 사용 설명서 (사용자)	94

INFORMATION

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INFORMATION

Last update: 2015-02-05

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Have yourself instructed by qualified personnel in the proper and safe use of the product.
- Please keep this document in a safe place.

The "C-Leg Protector" is referred to simply as the protector/product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

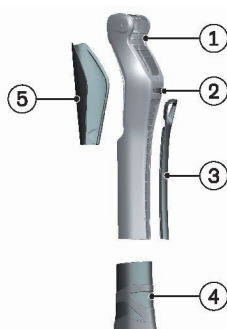
Have the qualified personnel instruct you in the use of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

2.1 Design

The product consists of the following components:

1. Protector main unit
2. Upper closure (one side always stays on the main unit)
3. Lengthwise closure
4. Foot cuff for protector
5. Shield insert

2.2 Function

The protector provides a means of enclosing the C-Leg electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing of the knee joint is protected against external influences by the protector, e. g. scratching and surface contamination. The protector is not required for the proper and safe functioning of the joint.

2.3 Combination Possibilities

Prosthetic components that are suited for combination with the product are listed below.

Knee Joint Combination Possibilities

Designation	Reference number
C-Leg	3C98-3/3C88-3

Prosthetic Foot Combination Possibilities

Prosthetic foot		Foot cuff		
Designation	Reference number	Size S	Size M	Size L
Triton	1C60	24-25	26-28	29-30
Triton Vertical Shock	1C61	24-25	26-28	29-30
Triton Low Profile	1C63	24-25	26-28	29-30
Triton Heavy Duty	1C64	24-25	26-28	29-30
Trias	1C30	23-25	26-28	29-30
C-Walk	1C40	24-25	26-28	29-30
Dynamic Motion	1D35	23-25	26-28	29-30
Adjust	1M10	23-25	26-28	29-30
Axtion	1E56	23-25	26-28	29-31
Lo Rider	1E57	24-25	26-28	29-31

3 Application

3.1 Indications for use

The housing of the knee joint is protected against external influences by the protector, e. g. scratching and surface contamination.

3.2 Area of Application

The product is intended **exclusively** for C-Leg users.

3.3 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

4 Safety

4.1 Explanation of Warning Symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 Structure of the safety instructions

 CAUTION The heading describes the source and/or the type of hazard The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible: > E.g.: Consequence 1 in case of failure to observe the hazard > E.g.: Consequence 2 in case of failure to observe the hazard ► This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.
--

4.3 General Safety Instructions

NOTICE

Changes or modifications to the product made independently

Damage to the product.

- ▶ Have any changes or modifications to the product carried out only by authorised, qualified personnel.

CAUTION

Penetration of dirt and humidity into the prosthesis

Injury due to unexpected prosthesis behaviour or malfunction.

- ▶ Should moisture have penetrated the prosthesis, remove the product from the prosthesis and allow both to dry. Observe the safety information in the 646D790 C-Leg instructions for use.
- ▶ The product and the prosthesis must be inspected by an authorised Ottobock Service Centre.

5 Scope of Delivery

Scope of delivery for 4X860=*

- 1 pc. C-Leg Protector (without shield insert)
- 1 pc. 4P860=R lengthwise closure
- 1 pc. 4P860=U upper closure
- 1 pc. 4P880=* foot cuff for protector
- 1 pc. 647G1114 instructions for use (user)

The following components are not included in the scope of delivery and must be ordered separately:

Accessories

- 1 pc. 4P863=* shield insert

6 Handling

INFORMATION

Removing the foot cuff from the prosthetic foot is not possible. The foot cuff stays on the prosthetic foot even when the protector is not being used.

6.1 Protector Installation

NOTICE

Loss of stability due to inadequately installed retainers

No hold of the protector on the knee joint.

- ▶ Ensure that all retainers and closures are properly engaged and/or fastened.

- 1) Open or remove the closures.
- 2) Spread open the protector.
- 3) Thread the protector into the foot cuff (see fig. 1).
- 4) Put the protector over the knee joint and apply it.

INFORMATION: The retainers of the protector (at the side in the upper closure area) have to engage on the frame element of the knee joint (see fig. 2).

- 5) Squeeze the protector together with both hands.
- 6) Fasten the upper closure (see fig. 3).

- 7) Fasten the lengthwise closure by sliding it in (see fig. 4).

INFORMATION

Charging the C-Leg batteries with the protector installed

It may be difficult for the prosthesis user to open the charging receptacle cover through the recess in the lengthwise closure with his/her fingers. In this case, the battery charger plug may be used to pry open the cover.

6.2 Removing the Protector

- 1) Pull down the textile material of the foot cuff until the protector is exposed.
- 2) Open or remove the closures.
- 3) Spread open the protector and remove it from the knee joint.
- 4) Fasten the two closures after removing the protector.

6.3 Installing and removing the shield insert

- Slide the shield insert into the recess of the applied protector from anterior and let it engage (see fig. 5).

NOTICE

Loss of stability due to inadequately installed protector main unit

No hold of the shield insert on the protector.

- Ensure that the protector has been installed properly and that all closures are fastened.
- By opening the closures and removing the protector from the knee joint, the shield insert loses its hold and can be removed.

7 Maintenance

- During the standard inspection of the prosthesis components, also inspect the product.

7.1 Cleaning and Care

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- Only clean the product with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).
- 1) Clean the product with a damp, soft cloth.
 - 2) Dry the product with a soft cloth.
 - 3) Allow to air dry in order to remove residual moisture.

8 Legal Information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as class I accessories according to the classification criteria out-

lined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Appendices

9.1 Symbols Used

9.1.1 Symbols on the product

Legal manufacturer

Declaration of conformity according to the applicable European directives

 LOT PPPP YYYY WW

Lot number

9.1.2 Symbols on the shield insert

Legal manufacturer

Declaration of conformity according to the applicable European directives

 LOT PPPP YYYY WW

Lot number

9.2 Technical data

Ambient Conditions	
Storage and transport in original packaging	-25 °C/-13 °F to 70 °C/158 °F
Storage and transport without packaging	-10 °C/14 °F to 60 °C/140 °F max. 100% relative humidity, non-condensing
Operation	-10 °C/14 °F to 60 °C/140 °F max. 100% relative humidity, non-condensing
General information	
Reference number	4X860=* + 4P863=*
Product service life	Wear part subject to normal wear and tear
Protector weight (with closures)	450 g

General information	
Foot cuff weight	60 g
Shield insert weight	63 g

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Otto Bock Healthcare Products GmbH