
2C12

DE	Gebrauchsanweisung	4
EN	Instructions for use	9
FR	Instructions d'utilisation	15
IT	Istruzioni per l'uso	21
ES	Instrucciones de uso	26
PT	Manual de utilização	32
NL	Gebruiksaanwijzing	37
SV	Bruksanvisning	43
DA	Brugsanvisning	49
NO	Bruksanvisning	54
FI	Käyttöohje	59
PL	Instrukcja użytkowania	65
HU	Használati utasítás	70
CS	Návod k použití	76
RO	Instrucțiuni de utilizare	81
HR	Upute za uporabu	87
SL	Navodila za uporabo	93
SK	Návod na používanie	98
BG	Инструкция за употреба	103
TR	Kullanma talimatı	109
EL	Οδηγίες χρήσης	115
RU	Руководство по применению	120
JA	取扱説明書	127
ZH	使用说明书	132
KO	사용 설명서	137

1

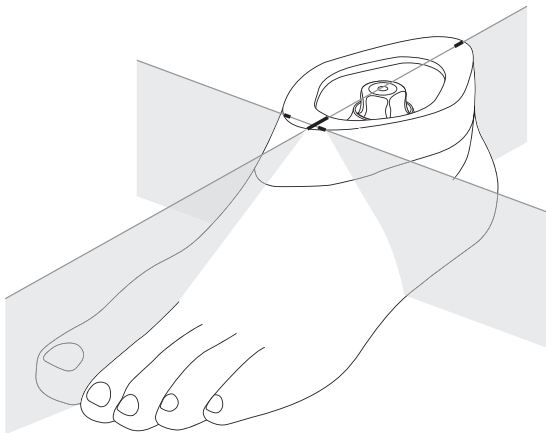2

3

4

5

6

INFORMATION

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INFORMATION

Last update: 2015-11-17

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

1.1 Construction and Function

The 2C12 footshell is a protective cover for a prosthetic foot. The footshell has a natural outer shape with an abducted big toe.

The toenails are **not** paintable.

1.2 Combination possibilities

Designation	Reference number
Prosthetic foot	Terion 1C10, Terion K2 1C11

2 Application

2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

2.2 Environmental conditions

Allowable environmental conditions	
Temperature range for use: -10 °C to +60°C	
Allowable relative humidity 0 % to 90 %, non-condensing	
Unallowable environmental conditions	
Mechanical vibrations or impacts	
Perspiration, urine, fresh water, salt water, acids	
Dust, sand, highly hygroscopic particles (e.g. talcum)	
Other environmental conditions	
UV light	Not resistant
Hydrolysis	Not resistant

2.3 Service Life

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination possibilities".
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION

Exceeding the service life and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service life").
- ▶ Only use the product for a single patient.

CAUTION

Contact with heat, embers or fire

Injuries (e.g. burns) caused by melting of the material

- ▶ Keep the product away from open flame, embers and other sources of heat.

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental Conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e. g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

NOTICE**Contact with salt water or water containing chlorine as well as abrasive substances (e. g. sand)**

Damage and premature product wear

- ▶ Following any contact with the substances identified above, promptly clean the product in accordance with the section "Cleaning".
- ▶ **Inform the patient.**

NOTICE**Mechanical damage to the product**

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

4 Scope of delivery

2C12=*				
Fig.	No.	Quantity	Designation	Reference number
–	–	1	Instructions for use	647G1092
2	–	1	Footshell	–
2	–	1	Connection cap	2C13=*
–	–	1	Toe insert	–

5 Preparation for use**⚠ CAUTION****Incorrect alignment or assembly**

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

NOTICE

Grinding the footshell

Premature wear resulting from damage to the footshell

- ▶ Do not grind the footshell.
- ▶ **Inform the patient.**

5.1 Applying/removing the footshell

Inserting the foot into the footshell

- > **Recommended tools:** 2C100 changing device for footshell
- 1) Pull the Spectra sock over the prosthetic foot, avoiding any wrinkles.
 - 2) Slide the prosthetic foot into the footshell. Insert the carbon spring into the slot on the toe insert (see fig. 3).
 - 3) Push the prosthetic foot into the heel area of the footshell until it locks into place (see fig. 5).
 - 4) Place the connection cap on the footshell (see fig. 6).

Removing the footshell

- > **Recommended tools:** 2C100 changing device for footshell
- 1) Push the heel area of the footshell to the rear and pull the prosthetic foot up.
 - 2) Remove the prosthetic foot from the footshell.

6 Use

⚠ CAUTION

Use of the product without shoes

Falling due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.
- ▶ **Inform the patient.**

6.1 Cleaning

⚠ CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only disinfect the product with the approved disinfectants.
- ▶ Observe the instructions for cleaning and care.
- ▶ **Inform the patient.**

NOTICE

Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.
- ▶ **Inform the patient.**

- > **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) Clean the product with clear fresh water and a pH neutral soap.
 - 2) Rinse the soap away with clear fresh water.
 - 3) Dry the product with a soft cloth.
 - 4) Allow to air dry in order to remove residual moisture.

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH