Otto Bock Support - 4024

(GB)

Function Stabilising wrist brace, permits free range of motion of the thumb. Easy application and suitable if weak pinch grip.

Indications Rheumatoid arthritis, osteoporosis

Material Darlexx® (polyamide, Lycra®, PU-film), plastic splint.

Application

- 1) Place the support on the wrist.
- 2) Close the velcro straps individually.
- 3) Adapt the thumb grip.
- 4) Make sure the brace is firmly fitted without too much pressure.
- 5) If necessary, adapt the plastic splint by heating it in hot water (75 °C – 80 °C/167 °F – 176 °F) and then reshape.

Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120 °C/248 °F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) Re: neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 6) If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 7) If you are allergic to heat, we advise you to use a product that not contains neoprene.

Washing Instructions

- 1) Close all Velcro.
- 2) Remove detachable splints.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry.

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Materiale Darlexx® (polyamid, Lycra®, polyuretanfilm), plastskinne.

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Indikation RA, osteoporos.

Material Darlexx® (polyamid, Lycra®, polyuretanfilm), plastskena.

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Indikation RA, osteoporose.

Material Darlexx® (Polyamid, Lycra®, polyuretanfilm), plastskinne.

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Indikation RA, Osteoporose.

Material Darlexx® (Polyamid, Lycra®, PU-film), Plastikschiene.

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