X-Soft 1.4 4X1=V1.4

DE Gebrauchsanweisung (Fachpersonal)	3
EN Instructions for use (qualified personnel)	17
FR Instructions d'utilisation (Personnel spécialisé)	29
IT Istruzioni per l'uso (Personale tecnico specializzato)	43
ES Instrucciones de uso (Personal técnico especializado)	57
NL Gebruiksaanwijzing (Vakmensen)	71
SV Bruksanvisning (Fackpersonal)	85
TR Kullanma talimatı (Uzman personel)	97

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1	Foreword	18
2	Product Description	18
2.1	Function	18
3	Application	18
3.1	Indications for use	18
3.2	Use/Field of Application	18
3.3	Conditions of use	18
3.4	Qualification	18
4	Safety	18
4.1	Explanation of Warning Symbols	18
4.2	Structure of the safety instructions	19
4.3	General Safety Instructions.....	19
5	Scope of Delivery	19
6	System Requirements.....	19
6.1	Minimum PC Hardware Requirements	19
6.2	Supported operating systems.....	20
6.3	Additional Recommended Software Components	20
6.4	Additional Installed Software Components.....	20
7	Installation	20
7.1	Continue Installation Without Registration	21
7.2	Continue Installation With Registration.....	22
8	Data Transfer between the Prosthesis and PC.....	23
8.1	Establishing a Bluetooth Connection to the Prosthesis	23
9	Starting the Program	23
10	Closing the Program	24
11	Uninstalling	25
12	Further Information	25
12.1	Information Regarding the Online Help	25
12.2	Obtaining Program Version Information	25
12.3	Software Updates	25
13	Troubleshooting	26
14	Trademarks	26
15	CE Conformity	26
16	Symbols on the product.....	27
	Licence Agreement	27

1 Foreword

INFORMATION

Last update: 2015-04-07

- ▶ Please read this document carefully.
- ▶ Follow the safety instructions and the precautions specified in this accompanying document.
- ▶ If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the Ottobock office responsible for your country.

The "4X1=V1.4 X-Soft 1.4" adjustment software is referred to simply as X-Soft, software or adjustment software below.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

2.1 Function

The "X-Soft" software is designed for the patient-specific adjustment of the products 3B1, 3B1-2, 3B5-X2, 3B5-X3, 3B5-2. The integrated alignment recommendation supports optimum adaptation of the component to the patient.

Communication between the software and the prosthesis takes place exclusively via Bluetooth data transfer.

3 Application

3.1 Indications for use

The "X-Soft" software is designed for the patient-specific adjustment of the products 3B1, 3B1=ST, 3B1-2, 3B1-2=ST, 3B5-2, 3B5-2=ST, 3B5-X2, 3B5-X2=ST, 3B5-X3, 3B5-X3=ST.

3.2 Use/Field of Application

The "X-Soft" adjustment software is intended for use exclusively with the products 3B1, 3B1=ST, 3B1-2, 3B1-2=ST, 3B5-X3, 3B5-X3=ST, 3B5-2, 3B5-2=ST.

The 60X5 BionicLink PC is the only permissible method of data transfer between the system components.

3.3 Conditions of use

- For the software to be able to be used, the PC must fulfil the system requirements.
- The battery of the prosthesis must be charged.

3.4 Qualification

The adjustment software may only be used by users who have participated in the relevant product training and have been certified for the application. Additional product training courses may become necessary to qualify for software updates.

4 Safety

4.1 Explanation of Warning Symbols

CAUTION Warning regarding possible risks of accident or injury.

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General Safety Instructions

CAUTION

Operator errors during the adjustment process with the adjustment software

Falling due to unexpected behaviour of the product.

- ▶ Participation in an Ottobock product training course is mandatory prior to using the product. Additional product training courses may become necessary to qualify for software updates.
- ▶ Correctly entering the foot size, the prosthesis dimensions and the body weight are important criteria for the quality of the fitting. If the values are too high, the prosthesis may not switch to the swing phase. If the values are too low, the prosthesis may trigger the swing phase at the wrong time.
- ▶ Always enter the weight in the specified unit.
- ▶ Ensure that the required dimensions are selected and entered correctly.
- ▶ If the patient uses walking aids (e.g. crutches or walking canes) during the adjustment process, readjustment is required as soon as the patient stops using these walking aids.

INFORMATION

- ▶ Please note that observing the prescribed system requirements is mandatory.
- ▶ It is recommended that you do not run any other programs during installation and use of the adjustment software, since this can affect performance.
- ▶ If you require further information, contact the Ottobock office responsible for your country.

5 Scope of Delivery

- 1 pc. "4X1=V1.4 X-Soft 1.4" software CD
- 1 pc. 647G1153 instructions for use (qualified personnel)

6 System Requirements

6.1 Minimum PC Hardware Requirements

The minimum hardware requirements depend on the operating system. The following requirements apply if they are higher:

For 32-bit platforms

- PC with an Intel Pentium IV processor
- 1 GB RAM
- At least 1.5 GB free hard drive space
- Graphics card with Open GL support
- Resolution 1024 x 768
- Display scale 100% or 96 DPI (higher settings can lead to display problems, depending on the resolution selected)

- 32-bit colour depth (16.7 million colours)
- CD-ROM drive or DVD-ROM drive
- 1 available USB port
- Mouse and keyboard

For 64-bit platforms

- PC with an Intel Core2 processor (x64) or faster
- 2 GB RAM
- At least 1.5 GB free hard drive space
- Graphics card with Open GL support
- Resolution 1024 x 768
- Display scale 100% or 96 DPI (higher settings can lead to display problems, depending on the resolution selected)
- 32-bit colour depth (16.7 million colours)
- CD-ROM drive or DVD-ROM drive
- 1 available USB port
- USB connection 2.0 or higher
- Mouse and keyboard

6.2 Supported operating systems

The following 32-bit operating systems are supported:

- Microsoft Windows 7, SP1 or higher, all editions except Microsoft Windows 7 Starter

The following 64-bit operating systems are supported:

- Microsoft Windows 7, SP1 or higher, all editions except Microsoft Windows 7 Starter
- Microsoft Windows 8

INFORMATION

Please note that the latest USB driver version must always be installed for the Bluetooth connection via the 60X5 BionicLink PC. You can download the latest driver version from the following site: <http://ottobock.com/en/datastation>. The latest version can also be downloaded via the "software update" function (see Page 25).

6.3 Additional Recommended Software Components

The following software components are recommended:

- Internet browser
- A MAPI-compatible e-mail system (e.g. Microsoft Outlook, Mozilla Thunderbird ...).

6.4 Additional Installed Software Components

The following additional software components are required and installed during installation of the software (if they are not already installed on the PC):

- Microsoft .NET Framework 4.5

INFORMATION

Before the installation is started, the installation program checks if there is sufficient free space on the hard drive and returns an error message if this is not the case. If this occurs, a corresponding amount of space must be freed up on the hard drive. All components required by the Ottobock software are copied to the hard drive automatically by the installation program.

7 Installation

Installation information

- Please note that observing the prescribed system requirements is mandatory.

- It is recommended that you do not run any other programs during installation and use of the adjustment software, since this can affect performance.
- You must have administrator rights to install the software.

Carrying out the installation

To run the setup program, start Microsoft Windows and insert the software CD into the PC's CD/DVD drive. The installation program starts automatically when the CD is inserted.

INFORMATION

If the installation does not start after the CD is inserted:

Windows 7/Windows 8: open "Computer" (key combination [Windows logo key +E]).

Right-click on the **CD drive** in the **right window**. Click the menu item "Install or run program from your media" with the **left mouse button**.

Follow the instructions of the installation wizard once the installation program has started.

Information: fields marked with an asterisk (*) are mandatory.

When the 'Registration' window is displayed, the following options are available to continue the installation:

- Continue installation without registration
- Continue installation with registration

7.1 Continue Installation Without Registration

If you do not want to register, click the '**Skip registration**' button and follow the further instructions of the installation wizard.

Once installation is complete, an icon for starting the Ottobock Data Station is added to the Windows desktop. The "X-Soft" software can be accessed after the Ottobock Data Station has been started.

The Ottobock Data Station is a superordinate software platform used to launch the corresponding programs. For this reason a separate icon for the "X-Soft" software is not created during installation.

7.2 Continue Installation With Registration

In order to register, enter the company information and then click the '**Continue**' button.

When the summary of the data already entered appears, the following possibilities (buttons) are available for registering the software:


Standard e-mail program available ("**Send registration**" button)

If you are able to send emails using a standard email program (e.g. Microsoft Outlook, Mozilla Thunderbird), carry out the following steps:

1. Click the "Send registration" button.
An email with the required registration data is sent automatically.

Webmail available ("**Copy to Clipboard**" button)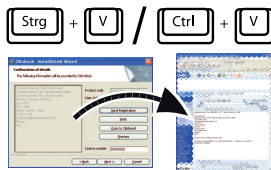

If you do not use a standard e-mail program, but have access to the Internet from your PC, carry out the following steps:

1. Click the "Copy to Clipboard" button.
2. Paste the contents of the clipboard into the text field of your email client (e.g. Gmail, Yahoo ...) with the key combination [Ctrl+V] or with the right mouse button.

Information: automatic processing is only possible in this way.

3. Send the e-mail to the following address:
registration.austria@ottobock.com

Fax available ("**Print**" button)

If you do not have an Internet connection, carry out the following steps:

1. Click the "Print" button to make a printed copy.
2. Send the printout via fax to the following address:
(+43-1) 526 79 85 • Otto Bock Healthcare Products GmbH • Kaiserstrasse 39 • 1070 Vienna • Austria.

Follow the subsequent instructions of the installation wizard.

Once installation is complete, an icon for starting the Ottobock Data Station is added to the Windows desktop. The "X-Soft" software can be accessed after the Ottobock Data Station has been started.

The Ottobock Data Station is a superordinate software platform used to launch the corresponding programs. For this reason a separate icon for the "X-Soft" software is not created during installation.

8 Data Transfer between the Prosthesis and PC

Prosthesis settings using the adjustment software can only be made via Bluetooth data transfer. In order to do so, a Bluetooth wireless connection must be established between the prosthesis and the PC with the help of the 60X5 BionicLink PC. To install the 60X5 BionicLink PC for the first time, follow the procedure described in the instructions for use supplied with this adapter.

INFORMATION

Do not disconnect the 60X5 BionicLink PC from the PC when there is an active Bluetooth connection.

INFORMATION

The adjustment software does not support the 60X4 BionicLink PC adapter.

8.1 Establishing a Bluetooth Connection to the Prosthesis

Upon delivery the Bluetooth function of the prosthesis is switched on.

The Bluetooth function can be switched off using the cockpit app or the remote control. When the Bluetooth function is switched off, it is only turned on for 2 minutes after connecting/disconnecting the battery charger and is then turned off again automatically. When a connection with the PC is active (the  symbol is lit up), the Bluetooth function is not switched off automatically.

INFORMATION

After the settings have been configured with the adjustment software, the program must be closed properly (see Page 24), otherwise Bluetooth remains on until the battery charger is connected and disconnected.

The following options are available to turn on the Bluetooth function for 2 minutes when it is turned off:

- 1) Connect/disconnect the battery charger or
 - 2) Hold the prosthesis with the knee head down (turn it upside-down), keep it still and wait for the feedback signals.
 - The Bluetooth function is activated for 2 minutes.
 - 3) During this time, a connection has to be established by clicking the **Connect** button.
- After successfully establishing a connection to the PC using the 60X5 BionicLink PC, the blue LED on the prosthesis and the blue LED of the BionicLink PC are flashing.


9 Starting the Program

INFORMATION

To run the software, ensure that the user is logged on to the PC with at least standard user rights.

- Double-click the  icon (Ottobock Data Station) on the desktop. The program starts.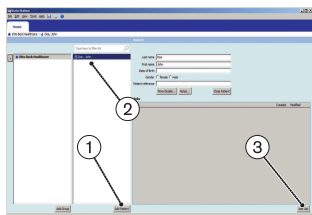

- After the Ottobock Data Station has been started the first step is to create a new patient record (click the "Add Patient" button) (item 1) or select an existing patient record (item 2).

Information: fields marked with an asterisk (*) are mandatory.

- Click the "Add Job" button (item 3).

- Click the "X-Soft" button. You will find this on the "Prosthetic Lower Extremity" tab.

- Before the first use, the "Username" and "Unlock-PIN" fields must be filled in with the relevant information that was given to you during the Ottobock product training course for the adjustment software.

Information: make sure to use correct spelling (upper/lower case) when entering the information.


- Click the "Log On" button.

- After entering the **unlock PIN** and clicking the "Log On" button you will be asked to enter and confirm a password of your own choice.

Information: the password entry window only appears after you have entered the unlock PIN. If you have already entered a password in the previous step, the password entry screen will not appear.

- Click the "OK" button.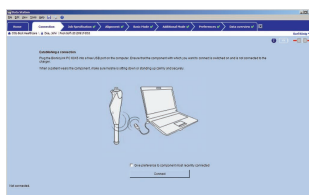

- Next, the screen for establishing the connection to the prosthesis appears.

- Click the "Connect" button to establish a connection with the prosthesis.

Compare the serial number that is displayed to the serial number of the prosthesis you want to connect to.

Information: The serial number is found on the inside of the knee joint frame.

INFORMATION

If the connection is disrupted during operation, see the section "Troubleshooting" (see Page 26).

10 Closing the Program

The following options are available for closing the program:

- Click "File > Exit" in the menu bar
- Click the "X" button in the top right corner of the program window
- Press the key combination [Alt+F4]

11 Uninstalling

To uninstall the Ottobock software, use the Windows uninstall function.

Windows 7

- 1) Open the Control Panel:
Click "Start > Control Panel > Programs and Features".
→ All installed software packages are displayed.
 - 2) Select the entry "Ottobock X-Soft".
 - 3) Click the "Uninstall" button.
- All components of the "Ottobock X-Soft" software will be removed from the hard drive.
However, the entered patient data and the Data Station are not deleted.

Windows 8

- 1) Press the key combination [Windows logo key +X].
 - 2) Click the "Programs and Features" entry.
→ All installed software packages are displayed.
 - 3) Select the entry "Ottobock X-Soft".
 - 4) Click the "Uninstall" button.
- All components of the "Ottobock X-Soft" software will be removed from the hard drive.
However, the entered patient data and the Data Station are not deleted.

12 Further Information

12.1 Information Regarding the Online Help

General Online Help

The online help can be accessed via the **F1** button or the "Help > Contents ..." menu bar.

Online help for the currently displayed page

Information on the page currently being displayed in the adjustment software can be accessed by clicking the  symbol.

12.2 Obtaining Program Version Information

The current program version of the installed software can be obtained from the menu bar. You should have this information available whenever you contact the software support.

- 1) Click "Help > About" in the menu bar.
→ A window with the currently installed software will open.
- 2) Click on the desired entry to obtain further information.

12.3 Software Updates

- 1) Click "Help > Otto Bock Support Website" in the menu bar when you are connected to the Internet.
→ The webpage for the "Data Station" software will open.
- 2) Search for the description of the software that is to be updated in the "Patch" column.
- 3) Search for the applicable version in the "Version" column.
- 4) Click "Download" in the column on the right in order to download the update.
- 5) Extract the "ZIP file" and execute it.

Software and driver updates will be made available for free at regular intervals.

13 Troubleshooting

Causes of connection errors (symbol appears)

If the  symbol in the top right corner is replaced by the  symbol, the connection to the prosthesis was severed.

Verify operational readiness of the 60X5 BionicLink PC and the prosthesis using the table below. Eliminate any causes that apply.

Event	Cause/required action
The blue LED on the prosthesis is not lit up	Prosthesis switched off Connect/disconnect the battery charger on the prosthesis to turn the prosthesis on.
	Bluetooth on the prosthesis is switched off Rotate the prosthesis by 180° (the sole of the foot has to face up) or connect/disconnect the battery charger to turn on Bluetooth for approx. 2 minutes. You must attempt to connect again during this time.
	Prosthesis battery drained Charge the prosthesis battery
The blue LED on the prosthesis is lit up continuously	Connection to the PC disrupted because the distance to the prosthesis is too large Reduce the distance between the prosthesis and the 60X5 BionicLink PC.
The green LED on the 60X5 BionicLink PC is not lit up	The 60X5 BionicLink PC is not operational - Check the connection of the BionicLink on the PC - Connect the BionicLink to a different USB port on the PC - If a hub is being used, connect the BionicLink directly to the USB port on the PC
The blue LED on the 60X5 BionicLink PC is not lit up	Connection to the PC disrupted because the distance to the prosthesis is too large Reduce the distance between the prosthesis and the 60X5 BionicLink PC.

Once the blue LED on the prosthesis and the green LED on the 60X5 BionicLink PC light up after eliminating the cause of the problem, start another connection attempt.

If a connection still cannot be established, contact the Ottobock branch responsible for your country.

14 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

Microsoft, Outlook and Windows are registered trademarks of Microsoft Corporation.

Pentium, Intel and Intel Core are registered trademarks of Intel Corporation.

15 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria out-

lined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

16 Symbols on the product

Legal manufacturer

Declaration of conformity according to the applicable European directives

Licence Agreement

Manufacturer: see back of this document

PLEASE READ THIS END USER LICENCE AGREEMENT ("EULA" OR "LICENCE AGREEMENT") CAREFULLY BEFORE YOU INSTALL OR USE THE OTTOBOCK SOFTWARE. BY INSTALLING OR USING THE OTTOBOCK SOFTWARE, YOU ARE AGREEING TO THE TERMS AND CONDITIONS OF THIS EULA.

DO NOT USE THE OTTOBOCK SOFTWARE UNLESS YOU AGREE TO THE TERMS AND CONDITIONS OF THIS LICENCE AGREEMENT. IF YOU DO NOT AGREE TO THE TERMS AND CONDITIONS OF THIS LICENCE AGREEMENT, YOU CAN RETURN THE OTTOBOCK SOFTWARE TO THE LOCATION WHERE YOU ACQUIRED THE SOFTWARE. IF YOU ACQUIRED THE OTTOBOCK SOFTWARE JOINTLY AND IN CONJUNCTION WITH HARDWARE, YOU CAN RETURN BOTH THE HARDWARE AND THE SOFTWARE.

IF YOU DO NOT AGREE TO THE TERMS AND CONDITIONS OF THIS EULA, YOU ARE NOT AUTHORISED TO INSTALL OR USE THE OTTOBOCK SOFTWARE.

1. Contractual partners and object of the agreement

1.1 By accepting this EULA, you are entering into a contract between yourself and the named Ottobock company; however, if you are acting on behalf of a company when installing or using the Ottobock software (e.g. as an employee of said company), the contract is concluded between the company and Ottobock (in which case "you" refers to said company). In case of any doubt, you are a party to the contract.

1.2 The licence covers the Ottobock software, including any related documentation (referred to as "Ottobock Software" below). The terms and conditions of this licence agreement also apply to any updates to the Ottobock software that replace or amend the original software unless such an update is the object of a separate licence agreement. Any possible maintenance of the Ottobock software or other Ottobock products is not covered by this agreement; a separate agreement is required.

1.3 To the extent that the Ottobock software will be used to provide recommendations regarding patient fittings (e.g. for certain adjustments to Ottobock products after taking customer measurements), please note that such suggestions merely represent non-binding recommendations from Ottobock. While Ottobock will take reasonable steps to ensure the suggestions are correct, it is not obliged to provide technically accurate recommendations. Accordingly, Ottobock is not liable for possible technical errors in the recommendations except in the case of intent or gross negligence.

Your own technical responsibility for correct patient fittings remains unaffected by the non-binding recommendations.

1.4 To the extent that training requirements are specified for the use of the Ottobock software, the software may only be used by qualified personnel.

2. Licence

2.1 The Ottobock software is protected by national copyright laws in many countries and by international treaties. Ottobock will prosecute any violation of the copyrights it holds. You are only granted the right to use the Ottobock software to the extent required by law or expressly granted to you according to this licence agreement. Ottobock and/or its licensors expressly reserve all ownership and/or other rights to the Ottobock software.

2.2 By accepting this EULA, you are granted the non-exclusive right to use the Ottobock software for your own purposes in accordance with its intended function. "Own purposes" refers solely to the purposes of the respective contractual partner. In the case of orthopaedic supply companies or other companies that provide patient services related to Ottobock products (e.g. prosthesis adjustments), "own purposes" in terms of this licence agreement are also the purposes of the respective patient who wants to use the Ottobock products in accordance with their intended use.

2.3 You do not have the right to modify, edit, translate, rearrange, or otherwise alter the Ottobock Software nor to decompile, disassemble, or reverse engineer the Ottobock Software yourself or via third parties unless expressly permitted to do so by law.

2.4 You are authorised to install the Ottobock Software on several computers at the same time, or to use it on a network where the software can be accessed on more than one computer simultaneously. Making the software accessible to the public is not permitted. For the purposes of this EULA, "computers" shall also be understood to mean those Ottobock products using the software according to its intended function.

3. Sublicences, assignment

Issuing sublicences for the use of the Ottobock Software is prohibited. Renting, leasing, loaning, or otherwise temporarily ceding the Ottobock software to a third party is also prohibited.

The permanent assignment of the usage rights granted to you is only possible when:

- the third party to whom the rights are to be assigned accepts the terms and conditions of this licence agreement,
- the entire Ottobock software, and not just parts thereof, is assigned, and
- you do not retain any copies of the Ottobock software, not even in the main memory or cache of your computer.

4. Liability

Ottobock is liable to you according to the provisions of the Product Liability Act; in case of intent or gross negligence; for the loss of life, physical injury or the impairment of health; where Ottobock issues any guarantees; and in other cases of mandatory legal liability – respectively pursuant to the applicable legal regulations. Damages shall be limited to an amount representing typically foreseeable damages. Damage compensation claims against Ottobock in case of slight negligence are excluded.

Any other claims for compensation by you against Ottobock – no matter on what legal grounds and especially due to material defects, defective title and/or the breach of other obligations arising from the contract on the part of Ottobock, its legal representatives, employees, or assistants or due to illegal acts – are excluded.

To the extent that the liability of Ottobock is limited or excluded by the preceding provisions, the same applies to the personal liability of the legal representatives, employees or vicarious agents of Ottobock.

The preceding provisions do not affect the burden of proof to your detriment. Claims for compensation on your part expire within the legal period of limitation, starting at the beginning of said period as established by law.

5. Applicable law and jurisdiction

This licence agreement shall be subject to the laws of the country in which the manufacturer is domiciled. The UN convention on contracts for the international sale of goods does not apply. The exclusive jurisdiction for all disputes arising from or in relation to this licence agreement is the registered office of the manufacturer.

6. Severability clause

Should one or more provisions of this agreement be or become ineffective or infeasible in whole or in part, the validity of the remaining provisions of the agreement shall not be affected.

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This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

A series of horizontal lines for writing, spanning the width of the page.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Otto Bock Healthcare Products GmbH
Brehmstraße 16 · 1110 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com