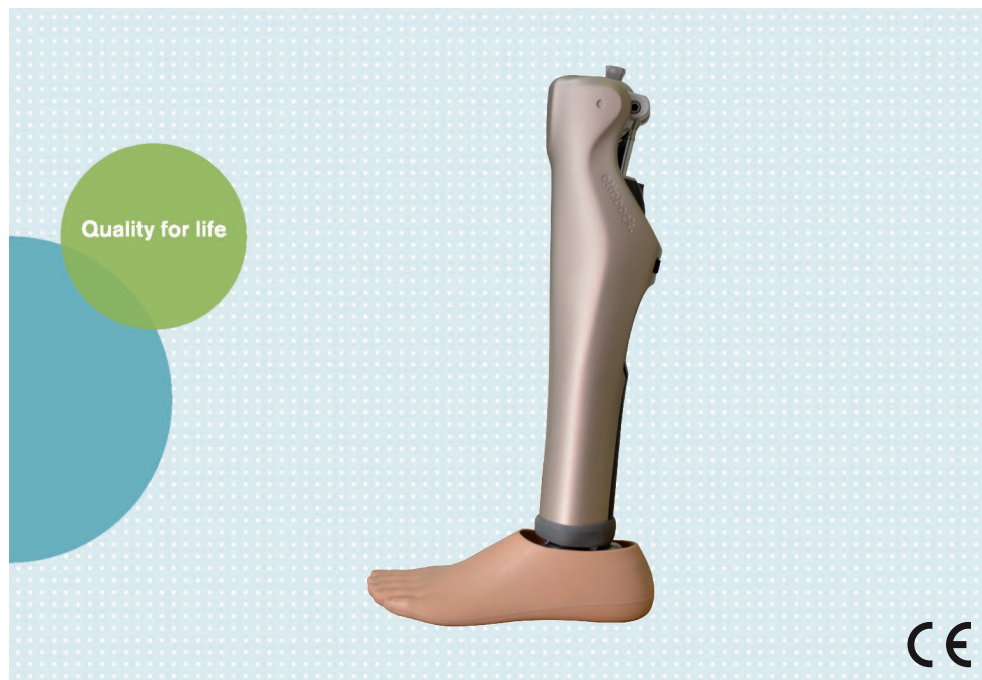
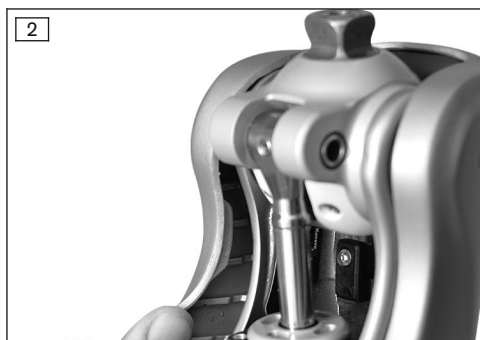
Kenevo Protector 4X840

DE	Gebrauchsanweisung (Benutzer)	3
EN	Instructions for use (user)	7
FR	Instructions d'utilisation (Utilisateur)	11
IT	Istruzioni per l'uso (Utilizzatore)	15
ES	Instrucciones de uso (Usuario)	20
PT	Manual de utilização (Usuário)	24
NL	Gebruiksaanwijzing (Gebruiker)	28
SV	Bruksanvisning (Användare)	33
DA	Brugsanvisning (Bruger)	37
NO	Bruksanvisning (Bruker)	41
FI	Käyttöohje (Käyttäjä)	45
PL	Instrukcja użytkowania (Użytkownik)	49
HU	Használati utasítás (használó)	53
CS	Návod k použití (Uživatel)	58
SK	Návod na používanie (Užívateľ)	62
TR	Kullanma talimatı (Kullanıcı)	66
RU	Руководство по применению (Пользователь)	70

INFORMATION

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INFORMATION

Last update: 2015-07-08

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Receive instruction in the correct and safe use of the product by qualified personnel.
- Please contact the qualified personnel if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- Please keep this document in a safe place.

The "Kenevo Protective Cover" is referred to simply as the protective cover / product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

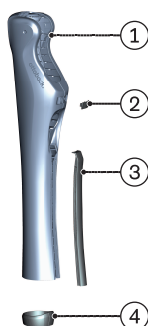
Have the qualified personnel instruct you in the use of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

2.1 Design

The product consists of the following components:

1. Protective cover main unit
2. Upper closure (one side always stays on the main unit)
3. Lengthwise closure
4. Flexible cover cap

2.2 Function

The protective cover provides a means of enclosing the Kenevo electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing of the knee joint is protected against external influences by the protective cover, e.g. scratching and surface contamination. The protective cover is not required for the proper and safe functioning of the joint.

2.3 Combination Possibilities

Prosthetic components that are suited for combination with the product are listed below.

Knee Joint Combination Possibilities

Designation	Reference number
Kenevo	3C60/3C60=ST

3 Application

3.1 Indications for use

The housing of the knee joint is protected against external influences by the protector, e. g. scratching and surface contamination.

3.2 Area of Application

The product is intended **exclusively** for wearers of the Kenevo knee joint.

3.3 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

4 Safety

4.1 Explanation of Warning Symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 Structure of the safety instructions

 **CAUTION**

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General Safety Instructions

 **NOTICE**

Changes or modifications to the product made independently

Damage to the product.

- ▶ Have any changes or modifications to the product carried out only by authorised, qualified personnel.

CAUTION

Penetration of dirt and humidity into the prosthesis

Injury due to unexpected prosthesis behaviour or malfunction.

- ▶ Should moisture have penetrated the prosthesis, remove the product from the prosthesis and allow both to dry. Observe the safety information in the instructions for use for your prosthesis as well.
- ▶ The product and the prosthesis must be inspected by an authorised Ottobock Service Centre.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

5 Scope of Delivery

- 1 pc. Kenevo Protective Cover main unit
- 1 pc. 4P840=R lengthwise closure
- 1 pc. 4P860=U upper closure
- 1 pc. 4P8 flexible cover cap
- 1 pc. 647G1138 instructions for use (user)

6 Handling

INFORMATION

Removing the flexible cover cap from the prosthetic foot is not possible. The flexible cover cap remains loose on the prosthetic foot when the protective cover is not being used.

6.1 Protector Installation

NOTICE

Loss of stability due to inadequately installed retainers

No hold of the protector on the knee joint.

- ▶ Ensure that all retainers and closures are properly engaged and/or fastened.

- 1) Open or remove the closures.
- 2) Spread open the protective cover.
- 3) Put the protective cover over the knee joint and apply it (see fig. 1).

INFORMATION: The retainers of the protective cover (at the side in the upper closure area) have to engage on the frame element of the knee joint (see fig. 2).

- 4) Squeeze the protective cover together with both hands.
- 5) Fasten the upper closure (see fig. 3).
- 6) Fasten the lengthwise closure by sliding it in.
- 7) Align the two lengthwise grooves on the flexible cover cap with the lengthwise closure (see fig. 4).

6.2 Removing the Protector

- 1) Pull down the flexible cover cap until the protective cover is exposed.

- 2) Open or remove the closures.
- 3) Spread open the protective cover and remove it from the knee joint.
- 4) Fasten the two closures after removing the protective cover.

7 Cleaning and Care

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

8 Legal Information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as class I accessories according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Appendices

9.1 Symbols Used

9.1.1 Symbols on the product

Legal manufacturer

Declaration of conformity according to the applicable European directives

LOT/PPPP YYYY WW

Lot number

9.2 Technical data

Ambient conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +40 °C/+104 °F max. 93% relative humidity, non-condensing

General	
Reference number	4X840
Product service life	Wear part subject to normal wear and tear
Protective cover weight (with closures)	380 g
Flexible cover cap weight	11 g

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Otto Bock Healthcare Products GmbH