50A3

(D)	Epi Forsa Plus Epicondylitisorthese	4
(GB)	Epi Forsa Plus Epicondylitis Orthosis	7
(F)	Epi Forsa plus: Orthèse (coudière) pour épicondylites	10
(I)	Ortesi Epi Forsa Plus per epicondilitis	14
(E)	Epi Forsa Plus Órtesis para epicondilitis	17
(P)	Epi Forsa Plus Ortótese para epicondilitis	20
(NL)	Epi Forsa Plus Epicondylitisorthese	24
(S)	Epi Forsa Plus Epicondylitisortos	27
(DK)	Epi Forsa Plus epikondylitortose	30
(PL)	Epi Forsa Plus orteza przedramienia przy zapaleniu nadkłykcia kości ramiennej	33
(H)	Epi Forsa Plus epicondylitis ortézis	37
(CZ)	Epi Forsa Plus epikondylární ortéza	40
(HR)	Epi Forsa Plus ortoza za epikondilitis	42
(TR)	Epi Forsa Plus Epikondilit Ortezleri	46
(GR)	Περιοδεσμός επιγονδύλιτιδας Epi Forsa Plus	49
(RUS)	Ортез при эпикондилите Epi Forsa Plus	52
(J)	Epi Forsa Plus 強化型肘関節支え	56
(中国)	Epi Forsa Plus 网球肘/高尔夫肘护具	59

links
left
gauche
sinistra
izquierdo
esquerdo
links
vånster
venstre
bal
levá
lijevo
sol

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1 Application

English

The 50A3 Epi Forsa Plus Orthosis is to be used **solely** as an epicondylitis orthosis and is only suitable for contact with healthy skin.

2 Indications and Effects

Epicondylitis humeri radialis ('tennis elbow'), epicondylitis humeri ulnaris ('golfer's elbow'). Specific indications must be diagnosed by the physician. The Epi Forsa Plus Epicondylitis Orthosis exerts targeted and controlled compression on the forearm muscles, to help reduce load on the irritated tendon insertions.

Attention!

In rare cases it is possible that wearing the orthosis will cause skin irritations in the area of the pad (e.g. with people who are sensitive to pressure or people who have sensitive skin). To prevent such irritations, we recommend that the Epi Forsa Plus not be worn in direct contact with the skin, and that a cotton stockinette or similar be worn under the orthosis.

3 Description and Application of the Orthosis

The initial fitting and application of the Epi Forsa Plus Epicondylitis Orthosis must be carried out by qualified personnel.

The patient is to be taught how to handle and use the Epi Forsa Plus Epicondylitis Orthosis.

The Epi Forsa Plus Epicondylitis Orthosis is made of skin-friendly cloth with a plastic insert. It is available in one universal size. It can be worn on the right or left forearm, simply by changing the position of the velcro pad (Fig. 1). The pad position for the left and right forearm is printed on the label and also indicated on the orthosis. The pressure of the adjustable pad can be altered by changing the tension of the continuously adjustable hook and loop closure.

To apply the orthosis, pull the hook and loop closure through the plastic loop, set the largest possible circumference and fasten the hook and loop closure (Fig. 2). Pull the orthosis over the concerned forearm. The end of the closure should point towards the outside (Fig. 3).

For *epicondylitis humeri radialis* you will find the correct pad position by making a fist with the concerned arm and flexing the extensor muscles. This tenses the muscle on the outside of the forearm on which the pad will be located.

For *epicondylitis humeri ulnaris*, you will find the correct pad position by flexing the hand of the concerned arm downwards. This tenses the muscle on the inside of the forearm on which the pad will be located.

Now position the pad on the respective muscle leaving about 4-5 cm (1.6-2 inch) of space between the elbow joint and the brim of the orthosis. Place the forearm on a firm surface and close the hook and loop closure towards the outside (Fig. 4 + 5). The orthosis should sit firmly, however, without constriction.

When handing over the orthosis to the patient, its correct fit must be checked.

4 Side Effects

Attention!

All aids – supports and orthoses – which are applied to the body may result in local pressure spots or, in some cases, even restrict the adjacent vessels or nerves when they are applied too tightly.

5 Contraindications

Attention!

In the case of the following clinical pictures, the aid should not be applied or worn without consulting a physician first:

1. Skin diseases and injuries in the area of the body where the aid is to be fitted, especially in cases of inflammatory symptoms. This also applies to scars with swelling, reddening and overheating.
2. Sensory and circulatory disorders of the forearm and hand
3. Lymphatic flow disorders, including unclear soft tissue swelling not near the area of the body where the aid will be applied.
4. Bottleneck syndrome (nerve compression in the arm).

6 Instructions for Use and Care

The orthosis is to be checked carefully each time before use to make sure it is functioning properly and has not suffered any damage or wear. Do not use the orthosis if any hook and loop closure or any other component of the orthosis shows excessive wear.

The patient should be instructed to see the physician immediately when he/she notices any unusual changes (e.g. increasing discomfort).

The daily duration of use and total period of application must be determined by the attending physician.

Improper changes to the orthosis are not permitted.

The product should not come into contact with grease or acidic agents, unguents and lotions.

Attention!

This product is not flame-resistant. The patient should be instructed to keep the product away from open flame or other sources of heat.

The orthosis without its pad may be washed at 30 °C / 86 °F with a standard mild detergent. We recommend closing the hook and loop closure before washing and using a washing bag to prevent any damage to it. The pad can be washed with mild soap and warm water (rinse thoroughly!). Air dry. Do not expose to the sun. Do not use oven or radiator heat for drying.

Instructions for Care

7 Restrictions of Use

The 50A3 Epi Forsa Plus Epicondylitis Orthosis is designed to be used on **one** patient only. The daily duration of use and period of application are dependent on the medical indication. This orthosis is designed for contact with intact skin only.

Attention!

Use of the orthosis by another person may represent functional or hygienic risks. Hygienic risks arise with the components that come into direct contact with the skin.

8 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

9 CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Otto Bock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock HealthCare GmbH